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Instruction sample collection Genotek swabs

Herewith you receive one or more Genotek swab(s). As of 1st April 2015, we use this new swab system for DNA collection. Due to technical progress one Genotek swab per animal is sufficient. Please use the instructions below in combination with the instruction on the packaging of the swab.

- We recommend - if possible - to rinse the mouth of the animal with water. To obtain the best result, do not allow the animal to eat half an hour before sampling.
- Open the sample collection bag carefully.
- Remove the swab from the sample collection bag. Brush with the swab about 2 times on each inside of the cheeks of the animal.
- Unscrew and remove the cap from the tube.
Do not pour the liquid out of the tube! Please leave it in place.
Be careful no contact between the liquid content of the tube and your skin occurs.
In case this accidentally happens, please rinse the skin surface thoroughly with water.
- Insert the flocked swab into the tube (into the liquid) and close the tube tightly.
- Take one Genotek swab for each animal.
- Make sure to clearly identify the animal on the label of the tube. For correct receipt it needs to be clear which swab originates from which animal.
- Please submit the samples to the address listed on the order confirmation of our webshop or the address listed on the sample submission form.

All samples must be accompanied with an order confirmation webshop or sample submission form.

For submitting samples, ordered through our webshop, a printed order confirmation will be sufficient. If you did not order a test online, please print a sample submission form, available on our website, www.certagen.de.